




# ErgoSafetyBar

## with Optional Towel Bar


Model #4006-SSB, 4006-SSP, 4006-MB, 4006-ORB

PW-94006-75BSS, PW-94006-75PSS, PW-94006-75BM, PW-94006-75BRZSS

### Owner's Manual

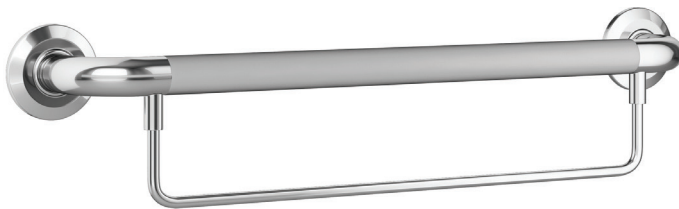

#### TOOLS REQUIRED FOR INSTALLATION

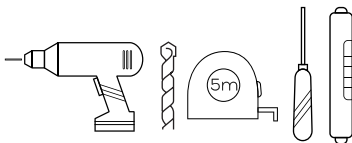

- Electric Drill
- 1/8" wood or metal drill bit
- 5/16" masonry drill bit
- Tape measure
- Screwdriver
- Level

#### IMPORTANT

Check local code for proper placement of ErgoSafetyBar to meet requirement such as ADA or ANSI 117.1 Consumer Safety Requirements.

**ERGOSAFETYBAR CAN BE MOUNTED THROUGH SHOWER WALL AND SCREWED DIRECTLY TO WOOD OR METAL STUD WHEN APPLICABLE.**


#### INSTALLATION QUESTIONS

Before returning your ErgoSafetyBar to the store or have any question about installation or missing parts please visit the CUSTOMER CARE area on our website [www.PulseShowerSpas.com](http://www.PulseShowerSpas.com).


# Installation Instructions

- 1 Position mounting flanges and mark hole locations.  
For wood or solid backing applications drill 1/8" pilot holes through shower wall into wood or metal stud.  
NOTE: If wall is tiled, drill a 1/8" pilot hole through tile then change to the a 5/16" diameter masonry tile drill bit.
- 2 Install screws through mounting flange and directly into wood/metal stud or bracing.  
Use masonry anchors if necessary.
- 3 Once the ErgoSafetyBar is securely fastened to wall, align cover plate locking tabs with slots in flange. Push cover against mounting until it locks.

## DETAIL MOUNTING INSTALLATION

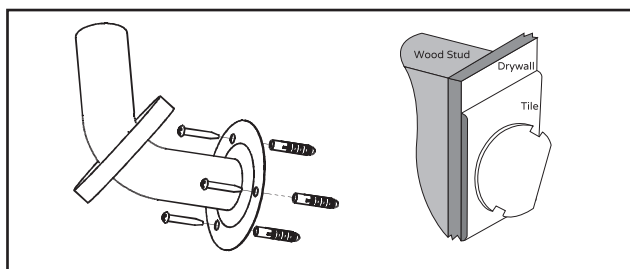

## INSTALLATION FOR OPTIONAL TOWEL BAR

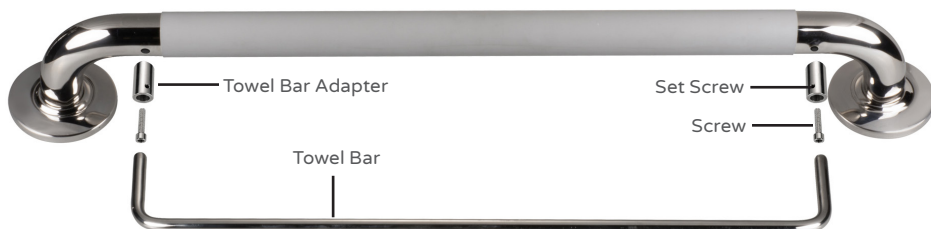

Towel Bar sold separately at [www.PulseShowerSpas.com](http://www.PulseShowerSpas.com)

