







Rollator Walker

Item # WB-2328

Rollator Walker Accessoriees

PHOTO	DESCRIPTION
	<p>1* Main Frame</p>
	<p>1* Backrest</p>
	<p>1 * Shopping Bag</p>
	<p>1 * Crutch Cup Holder</p>



1、 Brake handle

2、 Backrest

3、 Brake Line

4、 Side Pocket

5、 Seat

6、 Bag


7、 8 " Front wheel

8、 8 " Rear wheel

Rollator Walker Assembly



Step 1

As shown in the figure, open the main frame and then lay the cushion flat.

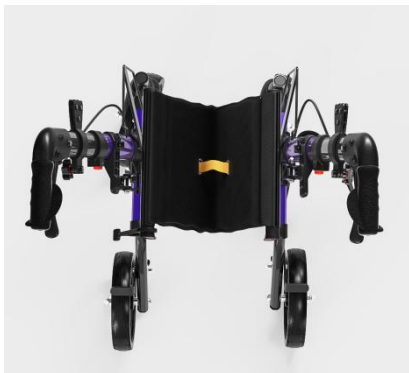


Step 2

Insert the front leg tube as shown in the picture and tighten the screws.

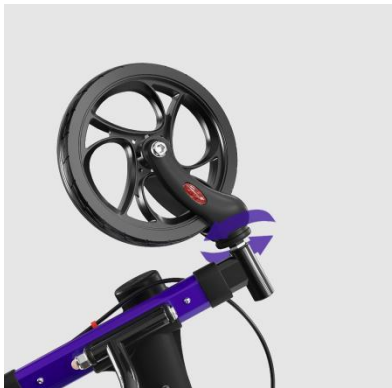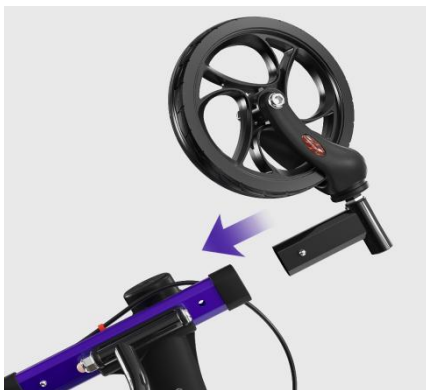


Step 3

Insert the rear leg tube as shown in the figure.

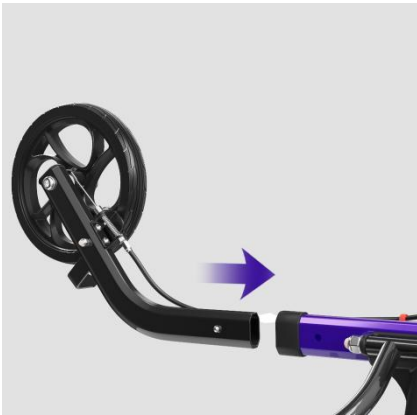


Step 4

Insert the armrest tube into the backrest clip,
and then insert it into the frame.

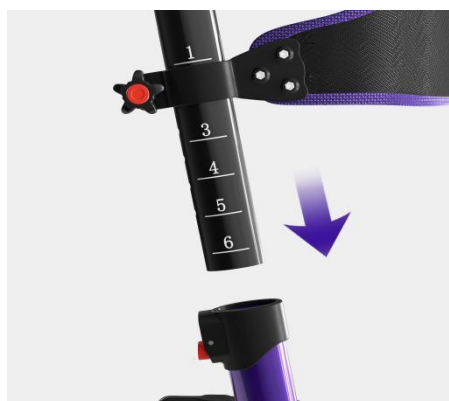

Step 5

As shown in the picture, insert the storage bag and press the crutch cup.

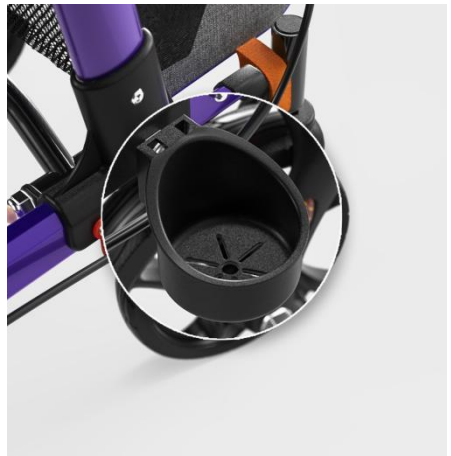

Once all the steps are completed the roller walker is assembled.

When the brake handle is opened downward, the rear wheels are braked.



WB-2328

Rollator Walker Introduction

1. Please make sure the Rollator is fully opened and the seat is down.
2. Be sure the hand grips are at equal height.
3. Always lock the brakes before sitting and do not use seat when unit is on an incline or uneven ground.
4. Before entering or leaving the chair, engage the dual wheel locks on both rear wheels.
5. Be sure that both knobs are fully tightened and are at equal heights. Test the Rollator for stability before applying weight to hand grips.
6. Regularly check the screws and fasteners on the Rollator and tighten if necessary.
7. Use bag only for transport of objects and do not overload. Maximum load: 11 lbs.
8. The maximum load of the rollator walker is 350 lbs.
9. Do not expose the product to temperatures above 104 ° F for an extended period of time, as this can cause the Rollator parts to overheat and burn the user.

Maintenance

Recommends periodic visual inspection of this product by the user, to ensure that all parts and hardware are secure, that components are in good working order and not worn, torn, frayed, or loose, and that there are no obstructions or impediments to normal, safe operation. If any part or mechanism appears to be unsafe or compromised, do not use the product.

If you experience problems, please send an email to our.

*Is a brand that focuses on the medical industry. Brand
Meaning: Bring health and goodluck to every customer.*