







Bring health and Good luck to our costumers

Rollator Walker

Item # WB-2212

Rollator Walker Accessoriees

Photo	DESCRIPTION
	1* Main Frame
	1 * Backrest
	2 * Front Wheel Bracket
	2 * Rear Wheel Bracket
	1 * Side Pocket

WB-2212



WEIGHT CAPACITY: 350 LBS

1 、 Brake line

2 、 Brake handle

3 、 Backrest

4 、 Seat

5 、 Side Pocket

6 、 Storage bag

7 、 Rear wheel

8 、 Front wheel

Rollator Walker Assembly

Step 1

As shown in the figure, open the main frame and lay the cushion flat.



Step 2

As shown in the figure, Insert the front wheel bracket into the main frame, then insert the screw and tighten the knob.



Step 3

As shown in the figure, Insert the rear wheel bracket into the main frame, then insert the screw and tighten the knob. (Note: The

hex screws need to be sunk into the hex holes and the wheels are on the outside of the frame)



Step 4

As shown in the figure, Insert the armrest bar into the main frame, then insert the screw and lock it tight. (Note: The hex screws need to be sunk into the hex holes)





Step 5

As shown in the figure, Press and hold the silver button on the backrest tube and insert it into the main frame.



Step 6

As shown in the figure, Hang the side pocket on the main frame, close the latch, and tie the Velcro to the main frame bracket.



Rollator Walker Introduction

1. Please make sure the Rollator is fully opened and the seat is down.
2. Be sure the hand grips are at equal height.
3. Always lock the brakes before sitting and do not use seat when unit is on an incline or uneven ground.
4. Before entering or leaving the chair, engage the dual wheel locks on both rear wheels.
5. Be sure that both knobs are fully tightened and are at equal heights. Test the Rollator for stability before applying weight to hand grips.

WB-2212

6. Please make sure the Rollator is fully opened and the seat is down.
7. Be sure the hand grips are at equal height.
8. Always lock the brakes before sitting and do not use seat when unit is on an incline or uneven ground.
9. Before entering or leaving the chair, engage the dual wheel locks on both rear wheels.
10. Be sure that both knobs are fully tightened and are at equal heights. Test the Rollator for stability before applying weight to hand grips.
11. Regularly check the screws and fasteners on the Rollator and tighten if necessary.
12. Use bag only for transport of objects and do not overload. Maximum load: 11 lbs.
13. The maximum load of the rollator walker is 350 lbs.
14. Do not expose the product to temperatures above 104 ° F for an extended period of time, as this can cause the Rollator parts to overheat and burn the