


**Power Lift Chair  
Assembly  
Instruction**

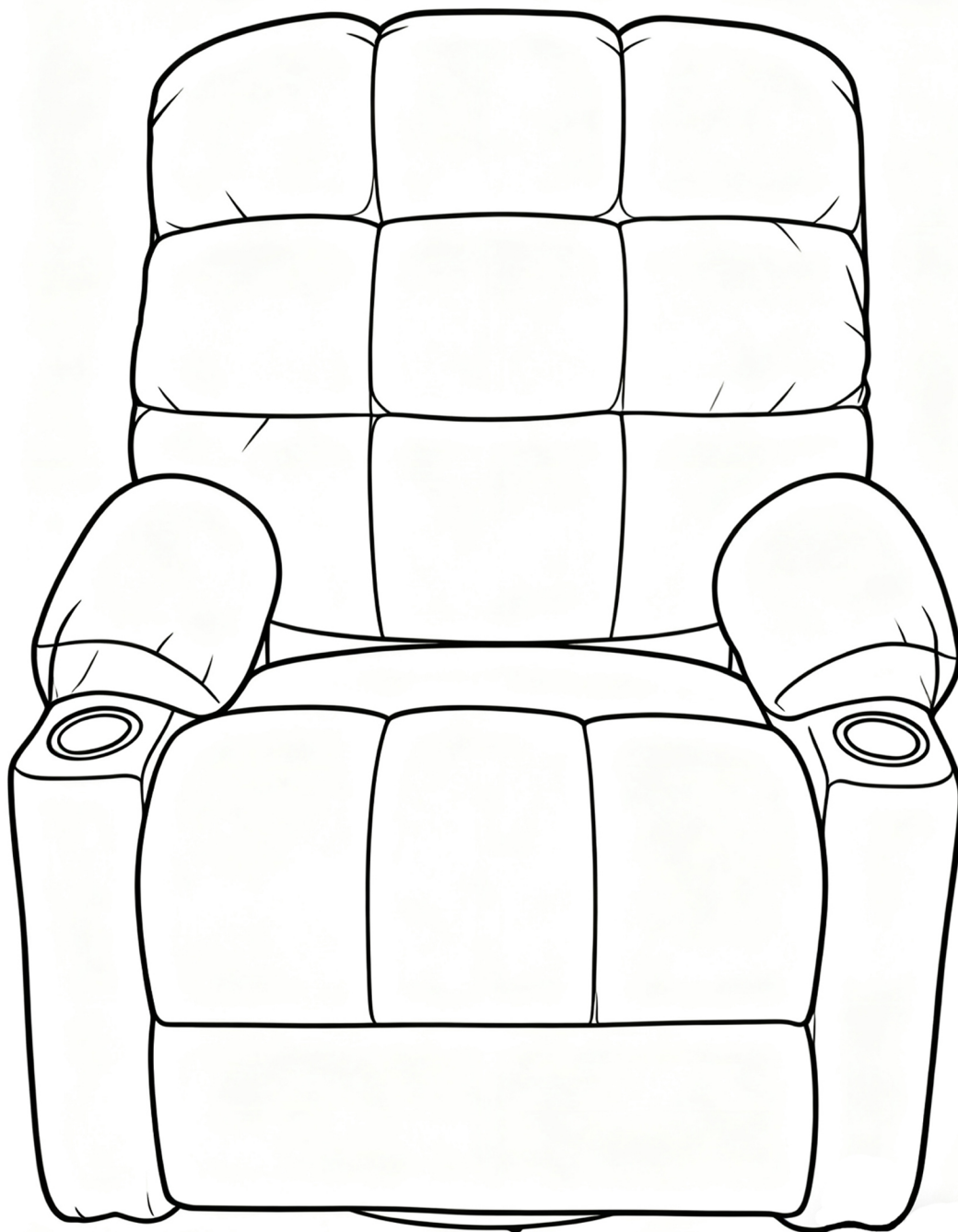

# FAQ

## Q1: Missing parts when received.

A: If you find some parts are missing from the box, please kindly send us messages to let us know the name of missing part regarding to the first page of instructions. We will send replacement to you asap.

## Q2: Some parts damaged when arrived.

A: Please send clear photos about damaged part to us as attachment, we will check and offer you best solution soon.

## Q3: The chair is stuck on lifting or reclining position, and no response when you press the lifting or reclining button.




A: Please kindly check if the power cord (F) are connected well with lift motor. Sometimes the power cord (F) may fall off from the motor.

If the connection is good, then check the lift motor if it has some anomaly, like some noise comes out when you press the lift button. If you can make sure the motor is defective, please contact us for replacement.


## How to remove the chair ?

Please note that you need to take apart the backrest before.


1. Force apart the buckles while take the backrest out from the arms.

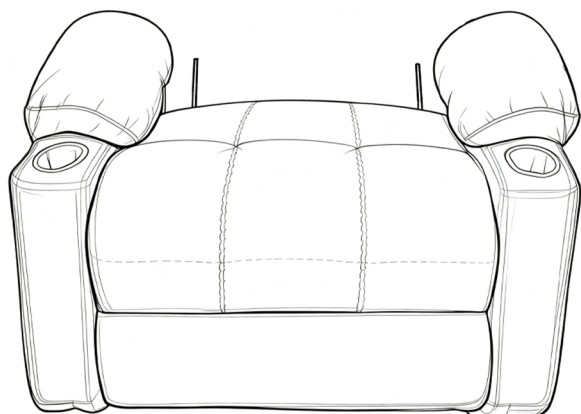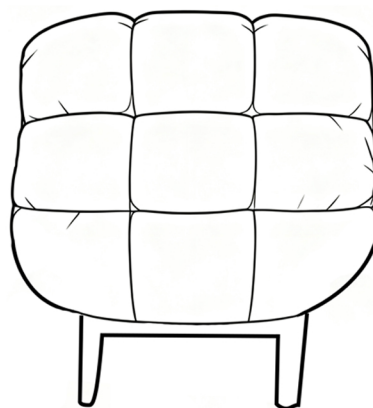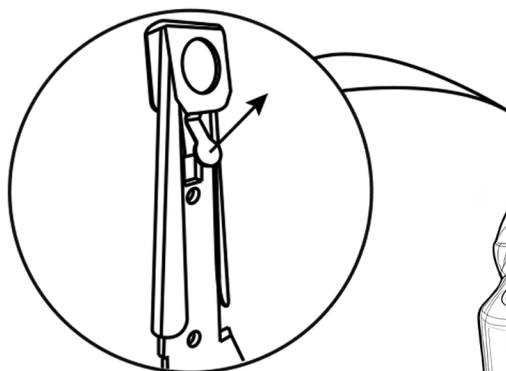


Please make sure you've got everything lists as following before assembly.

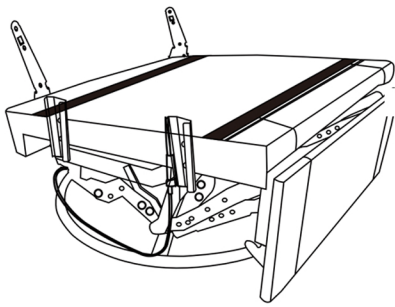


**A** Chair base

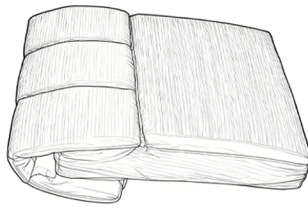

**B** Sofa back ear

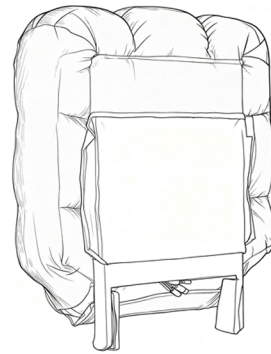


**C** Backrest

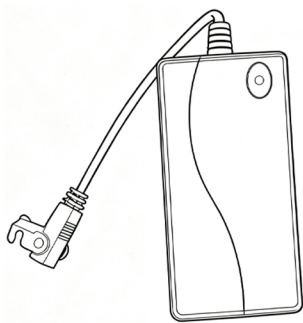


**D** Power cord




**E** Power cord for the base

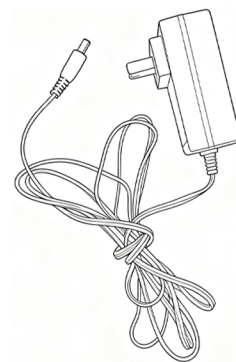


**F** Power Source

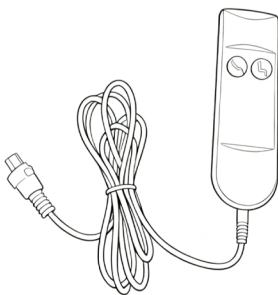


**G** Lift and lower remote control

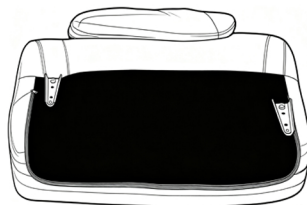

**H** Handrail

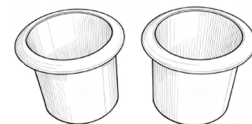


**I** Cup bucket

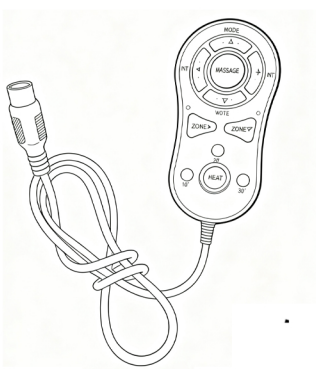


**J** Massage remote control


1. Take out the seat cushion and the seat frame, put the seat cushion in the corresponding position, and link the zipper with the seat frame

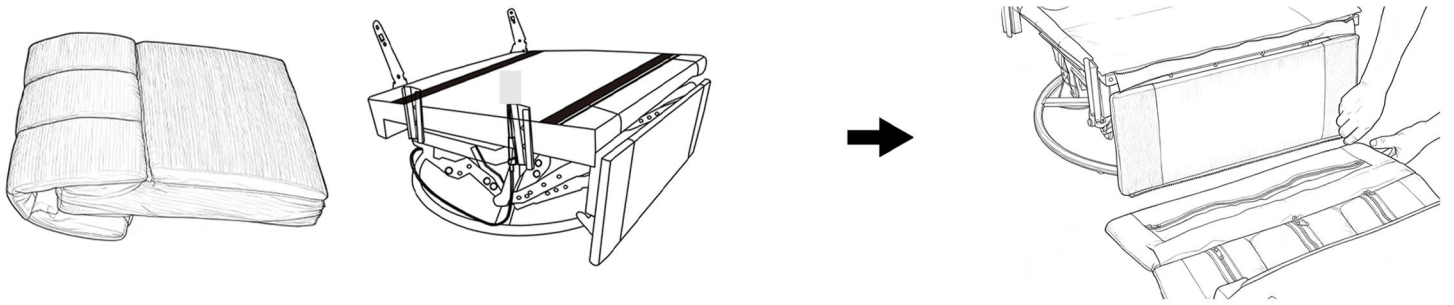


2. Connect plugs D and E, and then connect to wire No. 1. Connect the remote control G to line 2.

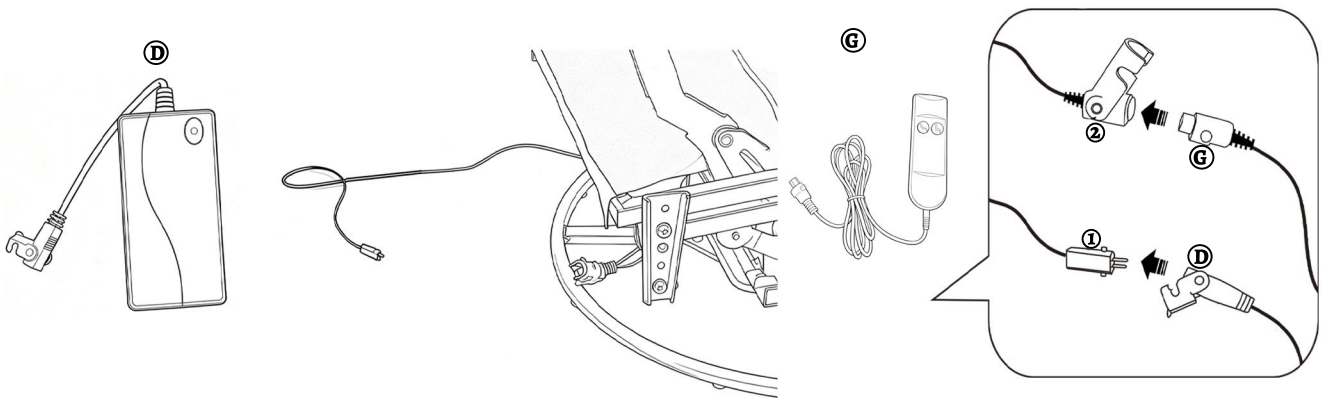


3. Press the remote control to open the stand, and fix the cushion and the foot stand with Velcro.

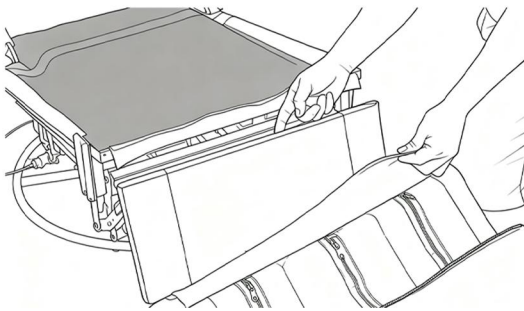


4. The front of the cushion is fixed with the bracket zipper, and then use the remote control to restore the bracket.

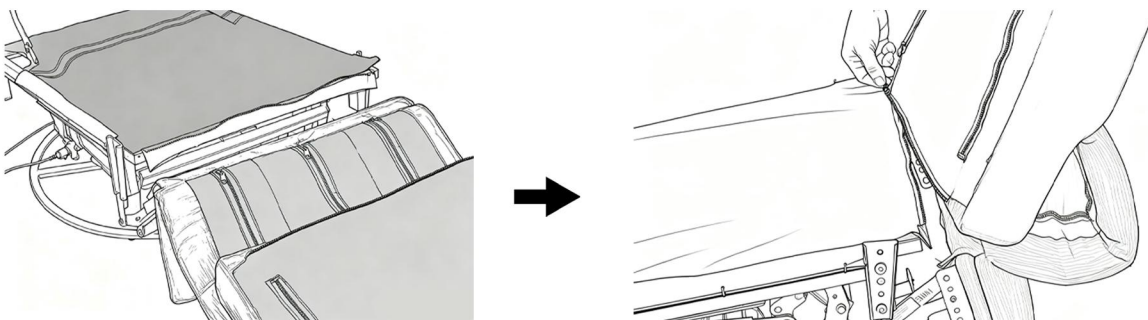


5. Buckle the armrests on both sides with the seat bracket.

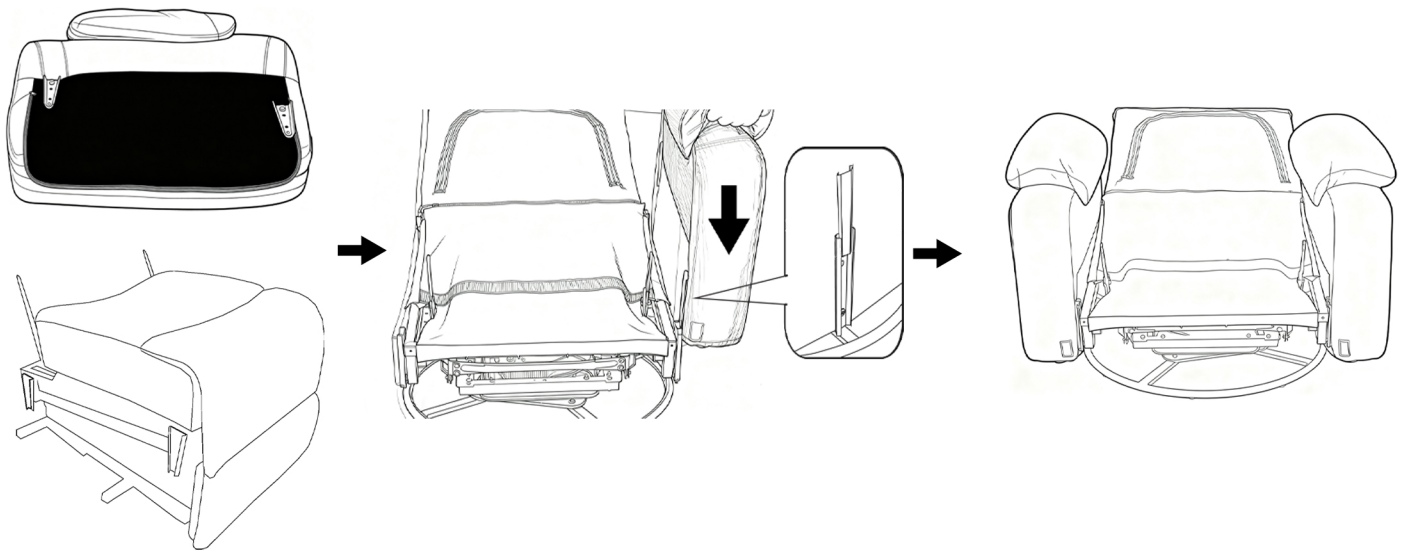


6. The cushion and bracket use Velcro fixed.

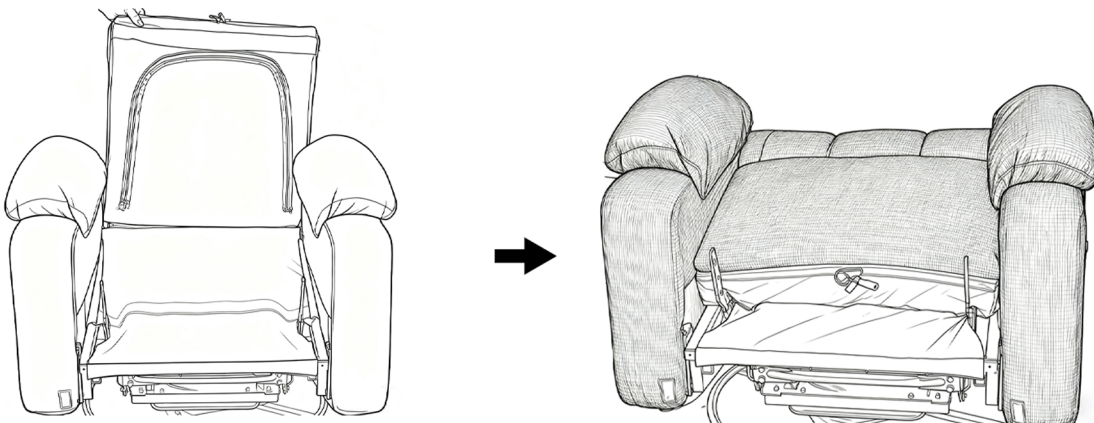


7. Buckle the backrest with the seat bracket

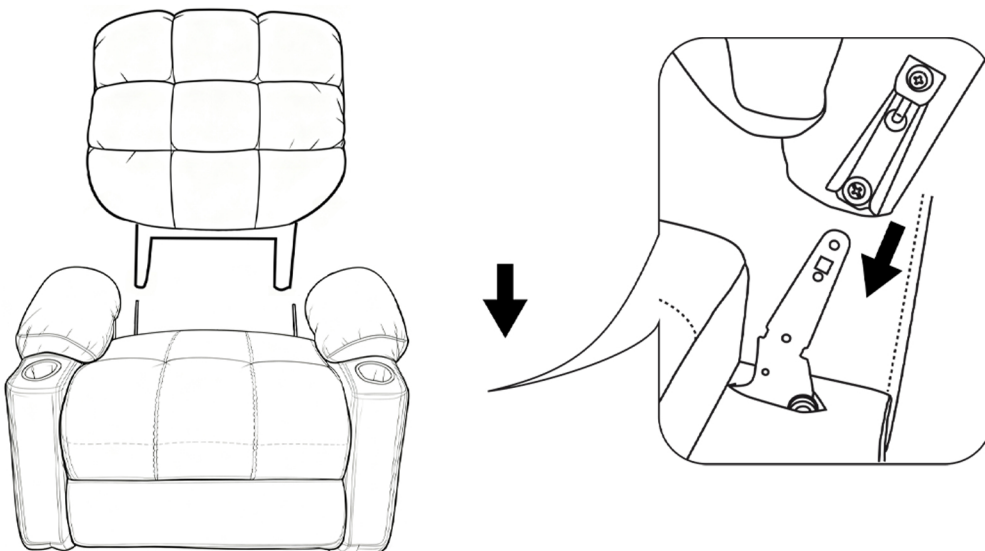

8. Plug the same color, insert the massage remote control and power cord.

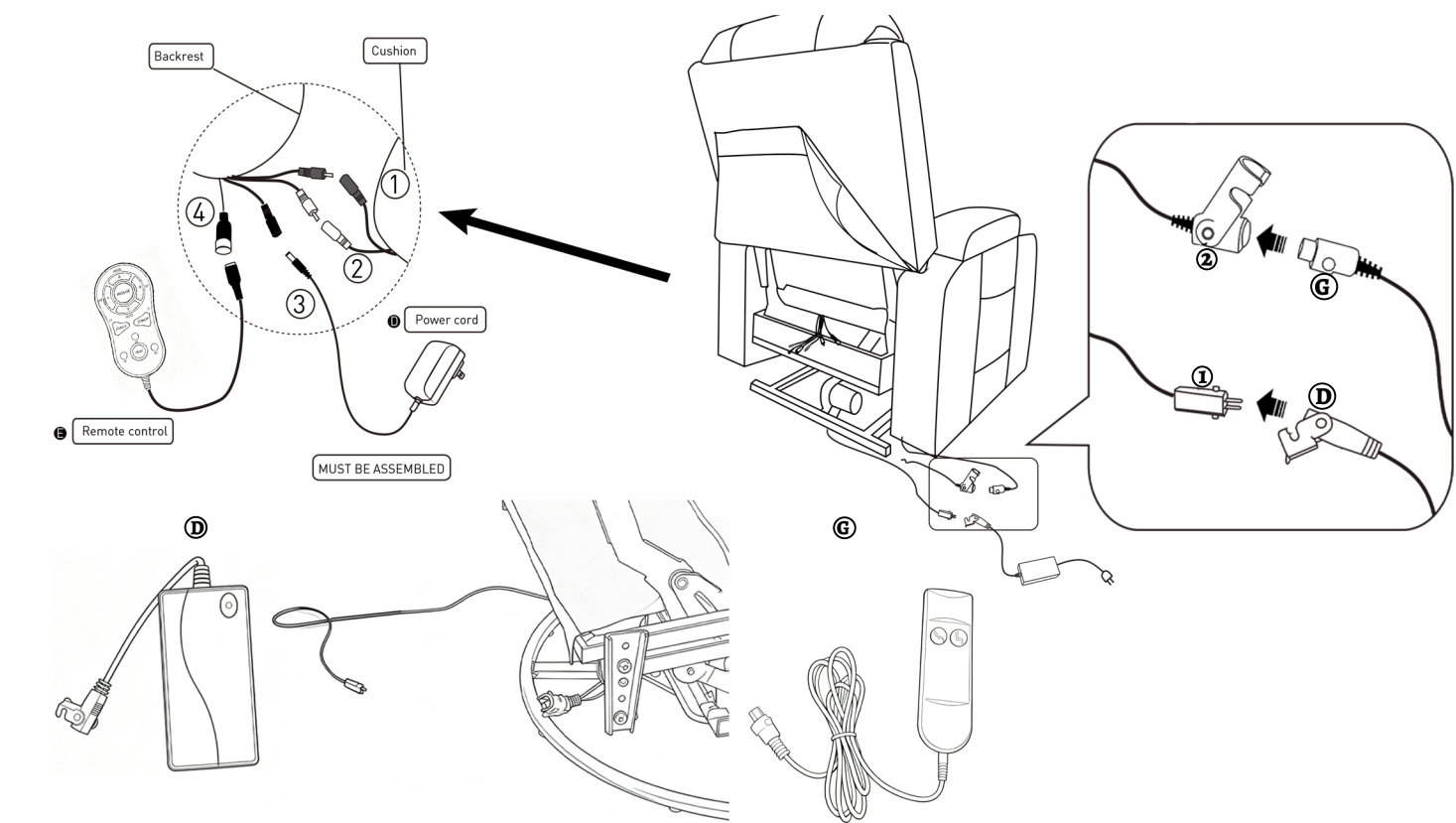

9. Fix the back cloth on the Velcro

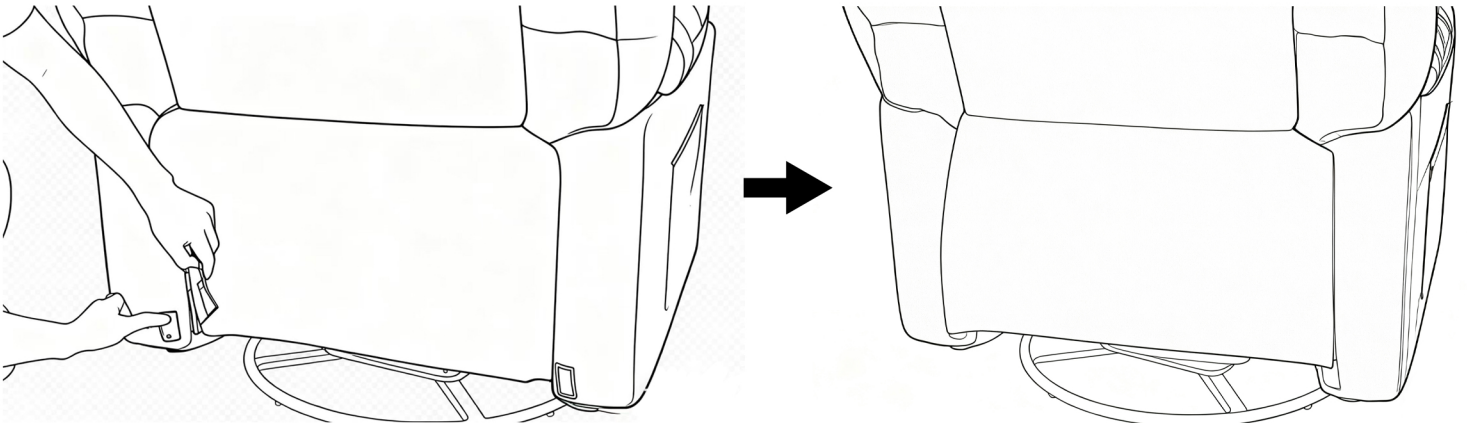

10. Insert the pail into the armrest

